


# Anmischanleitung Dual Rinse® HEDP

Anmischen unmittelbar vor Verwendung! Nach einer Stunde beginnt sich NaOCl in Gegenwart von HEDP abzubauen.

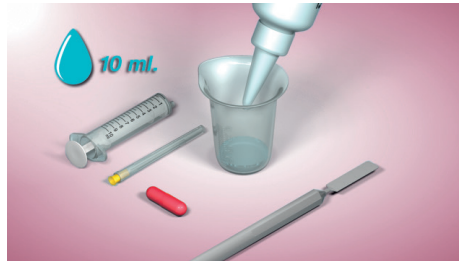

1. 10 ml dentale Natriumhypochlorit-(NaOCl) oder sterile Kochsalzlösung in sterilisierten Mischbecher gießen.
2. Dual Rinse® HEDP Kapsel vorsichtig über Mischbecher auseinanderziehen.



3. Pulver in Lösung rieseln lassen.
4. Mit sterilem Anmischspatel max. 2 Min rühren.



5. Klare Lösung in Spülspritze aufziehen.
6. Spülkanüle anbringen und 1 mm vor prospektiver Arbeitslänge abknicken, fertig!



Weitere Infos unter:  
[www.medcem.eu](http://www.medcem.eu)  
[www.medcem.ch](http://www.medcem.ch)

# Preparation of Dual Rinse® HEDP



Mix immediately before use! After one hour, NaOCl starts to degrade in the presence of HEDP.

1. Pour 10 ml of a dental sodium hypochlorite or physiological saline solution into sterilized mixing bowl.
2. Gently pull Dual Rinse® HEDP capsule apart.



3. Let powder trickle into the solution.
4. Stir using a sterile spatula for max. 2 minutes.



5. Draw clear liquid back into syringe.
6. Attach irrigating cannula, bend 1 mm short of prospective working length, done!



Find further information at:  
[www.medcem.eu](http://www.medcem.eu)  
[www.medcem.ch](http://www.medcem.ch)