


# X-Grey

Soft Smart Endo

**NIC<sup>®</sup>**



11mm miniature handle

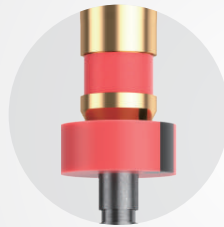

Radiopaque stopper



Precise depth marking



Continuous rotation



Debris free



Safe guiding tip



SHENZHEN SUPERLINE TECHNOLOGY CO., LTD.


— [www.nicdental.com](http://www.nicdental.com) —

## CHARACTERISTIC

- ① MaxTech 9.0 NiTi alloy
- ② Special cross section design
- ③ Archimedean spiral movement brings debris free
- ④ Super flexibility
- ⑤ Suitable for big curvature canals
- ⑥ Minimum invasive simple sequence



Size	21mm	25mm	31mm
X1	CFX 021 0X1	CFX 025 0X1	CFX 031 0X1
X2	CFX 021 0X2	CFX 025 0X2	CFX 031 0X2
X3	CFX 021 0X3	CFX 025 0X3	CFX 031 0X3
X4	CFX 021 0X4	CFX 025 0X4	CFX 031 0X4
X5	CFX 021 0X5	CFX 025 0X5	CFX 031 0X5
X1-X5	CFX 021 A01	CFX 025 A01	CFX 031 A01

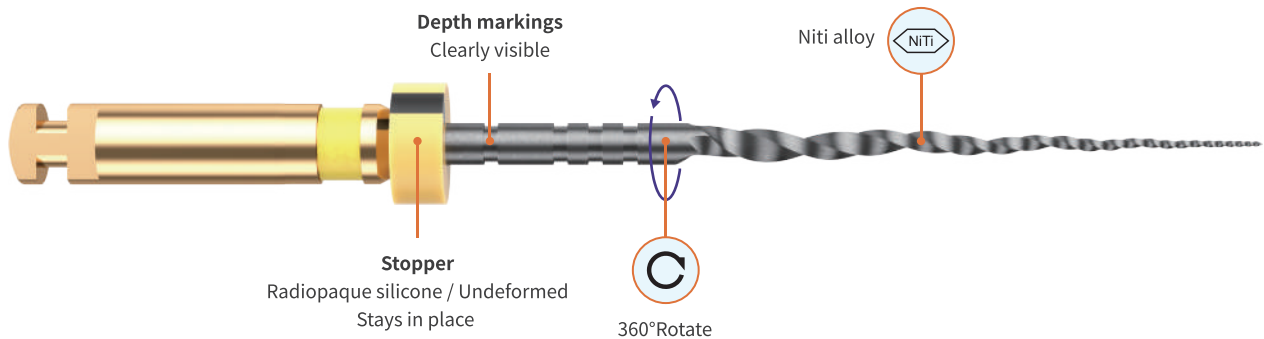

## OPERATING STEPS

X-Grey

SSK SSK  
#10 #15

+

SX T+  
019/.04 017/.02

+

X1 X2  
017/.04 025/.06

OPTIONAL

X3 X4 X5  
030/.07 040/.06 050/.05

X-Grry	SSK	SSK	SX	T+	X1	X2	X3	X4	X5
Size	#10	#15	019/.04	017/.02	017/.04	025/.06	030/.07	040/.06	050/.05
Speed(RPM)			300	300	600	600	600	600	600
Torque(N.CM)			3.0	1.5	3.0	3.0	3.0	3.0	3.0

SHENZHEN SUPERLINE TECHNOLOGY CO., LTD.

Add: Room 302, Nanshan Zhiyuan A4, No.1001 Xueyuan Avenue.  
Nanshan District, Shenzhen, China 518000

Tel: +86 755-2601 6156  
Fax: +86 755-2601 6026

E-mail: sales@supline.com  
Web: www.nicdental.com