- (EN) see [www.fkg.ch](http://www.fkg.ch) for additional information  
 (FR) informations supplémentaires sur [www.fkg.ch](http://www.fkg.ch)  
 (DE) weitere Informationen auf [www.fkg.ch](http://www.fkg.ch)

## RACE® EVO

Endodontic instruments  
 Instruments endodontiques  
 Endodontische Instrumente

FKG REF. 99.7AA.10.24A.XX - n°114 - 2019/12

## RACE® EVO

Description - Beschreibung

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## (EN) General information

### Parameters for use:

Recommended speed range: 800-1000 rpm continuous rotation.

Torque: 1.5 Ncm.

### Indications for use:

RACE® EVO endodontic instruments are intended for use in medical or hospital facilities, by qualified health professionals.

RACE® EVO instruments are designed for accessing, glide path and shaping of root canals.

RACE® EVO instruments are to be used with gentle strokes of 2-3 mm, applying very light apical pressure and allowing the instrument to progress passively along the canal. After 3 strokes, remove and clean the instrument, then irrigate; recapitulate with an ISO 010 hand file. *If necessary, a brushing motion may be applied to remove coronal interferences and/or ensure uniform shaping of irregular canals.*

### Contraindications:

Nickel-Titanium instruments must not be used on individuals with a known allergic sensitivity to these metals.

**Adverse reactions:** None known.

### Notice and precautions:

- Take several radiographs from different angles to determine the anatomy of the root canals (length, width and curvature).
- The number of canals to be treated with a RACE® EVO instrument is limited to a maximum of 8 root canals.
- Irrigate the canal thoroughly and frequently after the RACE® EVO instrument is extracted.
- Regularly inspect the RACE® EVO instrument during use, and discard if it shows any signs of wear (for example: straightening).
- If the RACE® EVO instrument is not progressing easily, remove it from the canal, clean it, inspect its cutting edges, then irrigate the canal and recapitulate with an ISO 010 hand file.
- Always inspect the instrument(s) before use, and discard (it) them if there are any visible defect(s).

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